


Export the MS9-5 / MS9-3 / MS9-3S datas to Excel

Export datas from MS9-5 / MS9-3 / MS9-3S

- 1- Format a CD / RW with the InCD software (RW is needed!)

Follow the indications starting from the icon InCD

Remove the auto selection of MRW Format

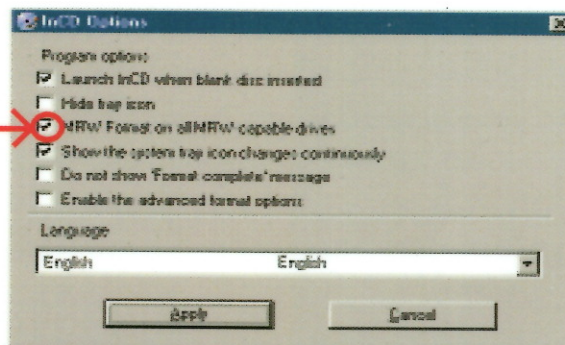

- 2- Open the MS9 software,
- 3- Select Analysis / Export,
- 4- Select the analyses you want to export, then click the «export» button;
- 5- Choose the text format in the Export dialog, then click OK,
- 6- On the «Save as» dialog box:
 - select the destination: [D:],
 - give a name to the file,
- 7- Click «Save» to start the export , then quit the program

Import the .txt datas to Excel program

- 1- From Excel software, select Menu / Files / Open
- 3- Select the .txt file you want to import,
- 5- Step 1: Choose «separate» for the kind of original datas
- 6- Step 2: Check the semi-colon symbol to separate the datas
- 7- Check the option to eliminate the too many consecutive symbols
- 8- Step 3: Select «Close»


MS9-5 / MS9-3 / MS9-3S: datas.mdb file backup

The process described below allows to copy the data.mdb file towards a CD/RW. If your machine needs to be reinstalled, following a technical problem, you will be able to recover the last backup «datas.mdb».


- 1- Format a CD / RW with the InCD software (RW is needed!)

Follow the indications starting from the icon InCD.


Remove the auto selection of MRW Format

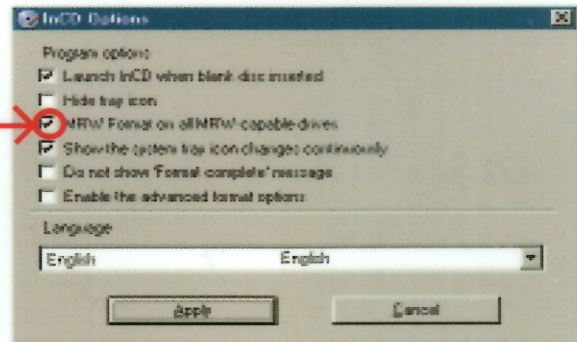

- 2- Open the MS9 software,
- 3- Select Analysis / Backup,
- 4- On the «Backup» dialog box:
 - Remove the «Delete» preselection (1),
 - Choose the good destination to [d:] (2),
 - You can change the filename ([datas.mdb] by default) (3),
 - Click the RUN button,

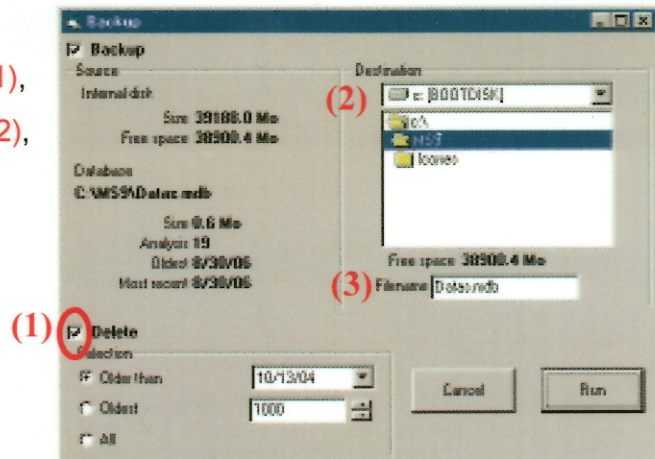

- 5- On the confirmation screen:
 - Click OK (4),
 - Wait until the process finishes,
 - Then, click the Quit button (5).

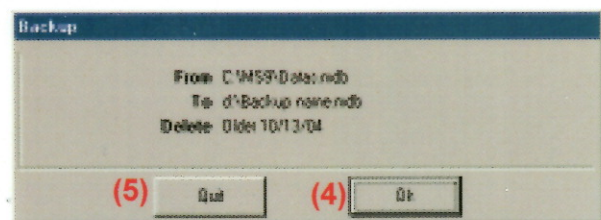

WARNING: Don't use the backup function to save data you want to use on Excel.

See another FAQ: MS9-53_Datas Export to Excel.

This datas.mdb file can only be used with MS9-5 / MS9-3 / MS9-3S.



DATE: 08/31/2006

File: MS9-53_datas file backup.pmd

MS9-5 / MS9-3 / MS9-3S: datas.mdb file backup

1/1

MELET SCHLOESING Laboratoires 9, Chaussée Jules César - Evolic 402 - 95520 OSNY - FRANCE

